Uro-Tainer® M

0,9% NaCl

PVC-free

BG	Информация за употреба	2
CZ	Informace pro použití	5
DE	Benutzerinformationen	8
DK	Oplysninger om brug	11
EE	Kasutusjuhend	14
EL	Πληροφορίες χρήσης	17
ES	Instrucciones de uso	20
FI	Tietoa käyttäjälle	23
FR	Mode d'emploi	26
GB	Information for Use	29
HR	Informacije za korisnika	32
HU	Használati utasítás	35
IT	Istruzioni per l'uso	38
NL	Informatie voor gebruik	41
NO	Informasjon om bruk	44
PL	Instrukcja obsługi	47
RO	Instrucțiuni de utilizare	50
SE	Bruksanvisning	53
SK	Informácie o použití	56

schwarz

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Lätus - - -

Irrigation Solution___190_GIF
190/NU19091N/1219
GIF (A5-Booklet, 19-spr.)
Standort Crissier

Font size: 7,0 pt.

Uro-Tainer® M 0,9 % NaCl

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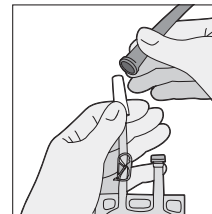
STERILE 

Uro-Tainer® M 0.9% NaCl

Maintenance solution for urethral and suprapubic catheters

1. What Uro-Tainer® are and what they are used for

Uro-Tainer® M 0.9% NaCl is a solution for intravesical instillation of medication such as antibiotics, disinfectants, cytotoxics, antimycotics, corticosteroids, etc, via an indwelling or single-use urinary catheter. They are sterile, endotoxin-free, ready-to-use medical devices in PVC-free bags.

2. What you need to know before Uro-Tainer® are used

Do not use Uro-Tainer®

- In case of hypersensitivity (allergy) to any of the ingredients of the solution (listed in section 6).
- In case of cystitis or other uro-genital condition that can produce haematuria (blood in the urine).
- Several days after surgery on the bladder or the urinary tract.

Warnings and precautions

- Talk to your doctor, pharmacist or nurse before using Uro-Tainer®.
- Under no circumstances should the NaCl 0.9% solution be injected or infused.
- Instill the NaCl 0.9% solution out of the Urotainer® bag by gravity feed, avoid force.
- Apply using aseptic technique. A port disinfection before use is recommended.
- Only to be used if the bag and the sealing cap are intact, and the solution is clear.
- This is a single-use system. After opening, it must be used and disposed of promptly. Re-use of such devices creates a potential risk of patient or user. It may lead to contamination and/or limited functionality of the device, which causes to injury, illness or death of the patient.
- Discard properly after use. If medication has been added, refer to the manufacturer's disposal instructions.
- If the solution leaks during the flushing process from the urethra, the catheter is not correctly

placed in the bladder. In that case the catheter needs to be replaced by a new one.

- If the solution is not flowing, it may be the catheter is bent or plugged. In that case the catheter needs to be replaced by a new one.
- Precautions should be taken when operating on patients with spinal injuries because of the risk of autonomic dysreflexia.

3. How to use Uro-Tainer®

- Uro-Tainer® are closed and sterile systems to be connected to the urinary catheter and work all according to the operating instructions on the next page.
- The Uro-Tainer® M sachet is supplied sterile and should not be removed from its overwrapping until required for use.
- If desired, bring the solution up to body temperature by immersing the wrapped sachet in warm water.
- The bag that contains the rinsing solution collects the liquid after rinsing of the catheter. Catheter rinsing and liquid collecting after rinsing are performed by gravity alone, by maintaining the device above and below the level of the bladder respectively.
- One Uro-Tainer® solution will be used for such an "in-and-out" application which will last each no longer than in average 5 min. This process is recommended 1 to 2 times per day.
- In general the daily use is only indicated for post-operative patients during a short period. In contrast to that, long-time catheterized patients have catheters for very long periods. In this case the catheter is only rinsed with Uro-Tainer® 2-3 times per week.
- If you use Uro-Tainer® more often than you should, no adverse events are expected.
- When adding medication, refer to the manufacturer's instructions and verify its compatibility with the NaCl 0.9% solution.

- When adding cytotoxic drugs, it is essential to refer to the manufacturer's instructions special precautions: contraindications, preparation, administration, etc.
- The reseal ability of the injection port membrane of the Uro-Tainer® M has been tested to ISO 15747 (Section 4.1.10) using a G21 (0.8 mm) needle. The leak-tightness of the port has therefore been proven in conjunction with needles with an external diameter of 0.8 mm (G21).
- Maximum additional injection volume for the Uro-Tainer® M 50 ml = 87 ml, for Uro-Tainer® M 100 ml = 37 ml.
- Product remaining in sachet after emptying: <1 ml for the Uro-Tainer M 50 ml, <1 ml for Uro-Tainer M 100 ml.

4. Possible side effects

NaCl 0.9 % solution is not known to cause any side effects.

5. How to store Uro-Tainer®

Keep the Uro-Tainer® out of the sight and reach of children.

Do not use the Uro-Tainer® after the expiry date which is stated on the outer plastic bag.

Do not refrigerate or freeze.

Store in the original package.

Only use clear solutions in undamaged bags.

After opening, the Uro-Tainer® must be used and disposed of promptly.

6. Contents of the pack and other information

What Uro-Tainer® contains

100 ml solution contains: Sodium chloride 0.9 g in Water for Injections

What Uro-Tainer® looks like and contents of the pack

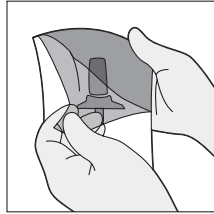
The solution appears as a clear and colorless liquid.

Uro-Tainer® are made of a soft bag free of poly(vinyl chloride) (PVC), in a protective plastic overwrapping with a separate latex-free injection port for adding medication, a flexible tube fitted with a clamp, and a sterile universal connector protected by a rigid plastic cap with a tamperproof seal.


Uro-Tainer® M 0.9% NaCl (2 Ports) are available in package sizes of 50 ml, and 100 ml.

This leaflet was last revised in 09/2016.


1. Tear open plastic bag and take out Uro-Tainer® M

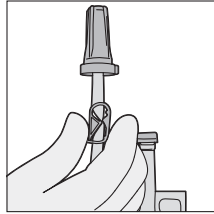
2. Ensure Uro-Tainer® M tube is closed by plastic clamp.

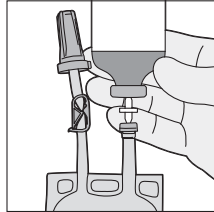
3a. Using aseptic technique, assemble transfer needle and vial onto the injection port. Squeeze saline into the vial from the Uro-Tainer® M. Release pressure from the Uro-Tainer® M to draw the contents of the vial into the sachet. The squeezing and release of air from the Uro-Tainer® M may be repeated several times until all the solution is safely transferred; or

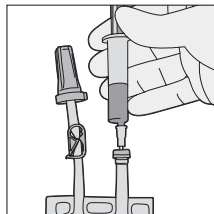
3b. Syringe some saline out of the Uro-Tainer® M through the injection port (from below) and into the vial to reconstitute the drug. Draw the reconstituted drug back into the syringe and inject into the Uro-Tainer® M again using the injection port (from above). If a liquid drug is used, syringe it directly into Uro-Tainer® M through the injection port.

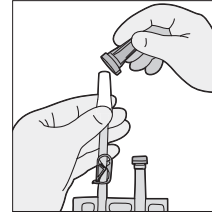
4. Remove tamper proof seal. Twist and withdraw cap without touching sterile connector.

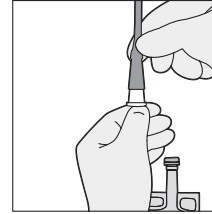
5. Insert connector into the catheter funnel.

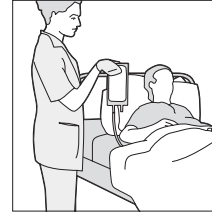6. Holding Uro-Tainer® M above the level of the bladder, release the clamp. Apply the solution by gravity alone.

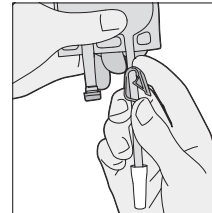
7. Close tube with clamp of the Uro-Tainer® M.

8. After the required period, release clamp and hold bag to have below the level of the bladder, drain the solution. Uro-Tainer® M can contain up to 150 ml and therefore allows to collect not only the irrigation solution but also eventual urine from the bladder.

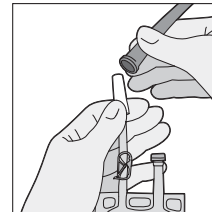9. Clamp tube and remove the connector from catheter by turning gently. Connect catheter to a sterile drainage bag.

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B. Braun Medical AG